


Hyper AIR® hybrid

- AIR-CELL MATTRESS -




WATCH NOW
on YouTube!

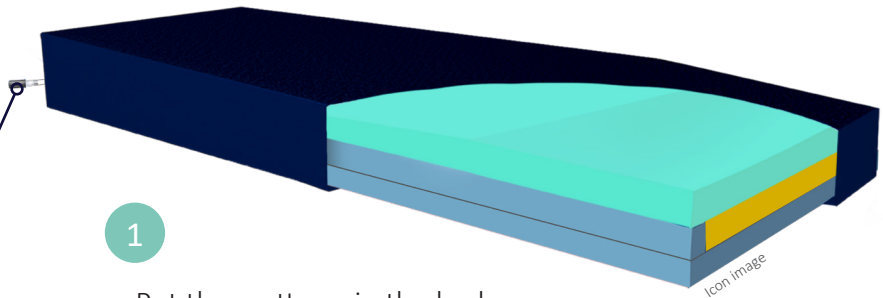

1

Put the mattress in the bed.
Please note the imprint on the top
of the mattress „THIS SIDE UP“.

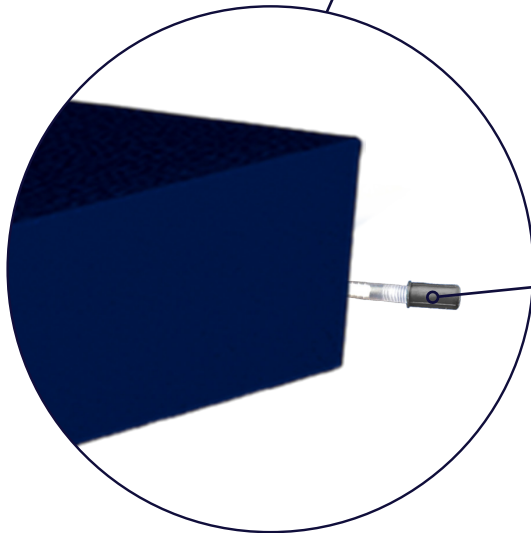

2

Open the valve by pulling and wait
approx. 5 minutes, until the air-cells
have inflated automatically.

3

Close the valve
by pressing.

(Valve lock cannot fall off)

4

Place the patient on the mattress.
Carefully open the valve by pulling the
grey cap, so that air escapes and carry
out a fingertest* at the same time.

5

Close the valve by
pressing.

6

Daily checking of the correct filling
of the air-cells is recommended.
An adjustment may be necessary.

*Place hand between buttocks and mattress.
Optimal lying comfort is achieved when
only a slight finger movement of 1 – 2,5 cm
is possible.